

OAKWORKS®

Nova



OAKWORKS®, Inc.
923 East Wellspring Rd., New Freedom, PA 17349
Toll Free Phone: 800-558-8850 Phone: 717-235-6807
Toll Free FAX: 877-562-4787 FAX: 717-235-6798
www.oakworks.com

Bodies Talk. We Listen.




All portable tables:
SlideLock™: U.S. Patent 5,009,170
Unlock: U.S. Patent 5,943,965
Patent Pending




Handcrafted in the U.S.A.

CAUTION
Dynamic Load Rating:
450 lbs.


15704713 NVL3272473PLTVN Set# NVS367130
B Loc SH Product Nova LS3 27 x 24-34 x 73 PL/TT/VN
TTCloud W 27 H 24-34 L 73 Foam PL 8/8/2007
Order# 500441 Line Notes

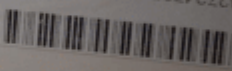

- To Close the Table:**
- Turn the table on its side with the rubber bumpers down. If you are using the table with the face rest, move it under the table by pushing the insertion down into the clip provided (Fig. 5).
 - Fold the legs into the table and make sure the armrest is lying flat and the cables are inside of the table. Push the two halves of your table together to lock (Fig. 6).
- For Tables with the Shiatsu Option:**
- With your table on its side, release the cables from the rings (Fig. 8a) and turn the unlock clasp to the side. (Fig. 8b) and lay the table against the underside and lay the table down flat on the floor (Fig. 9).
 - To return to normal height settings, see the separate CAUTION label that refers to the Shiatsu Cable Option.

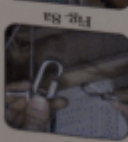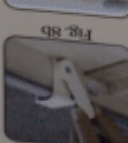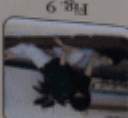

- To Set-up the Table:**
- Place the folded table sitting on the rubber bumpers, with the heel of your hand, bump the tab between the face rest outlets and the table will open. (Fig. 1)
 - Open the table up until the legs are fully extended. Remove the face rest. (Fig. 2)
 - When the table is fully opened, tilt the table upright until all four feet are on the floor. Lift one end of the table, press down on the middle and tension the cables a little. (Fig. 3)
 - Choose the hole that represents the desired height and after removing the knob, put the bolt through that hole. Replace and tighten the knob by hand until snug. Repeat on all other legs extensions, setting all extensions at same height. (Fig. 4)
- To Change the Table Height:**
- Choose the hole that represents the desired height and after removing the knob, put the bolt through that hole. Replace and tighten the knob by hand until snug. Repeat on all other legs extensions, setting all extensions at same height. (Fig. 4)

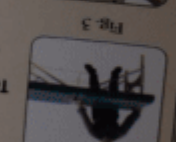

Nova